



# Effegi Brega – Test Results

Dental Products, Materials and Applications

---

01-12-2018 - Piero Rebecchi - Effegi Brega

# Surgical Guides


## Results:

- Cycles 8mins @ 20bar
  - 10 models tested
  - 0 broken (all cleaned)



## Increasing Statistics

- 3 more batches (30 models) were tested using 40bar
- Each batch included 10 models, all models are from sparse design
- Each batch lasted cycle 8mins
- Results:
  - All completely clean, 0 broken

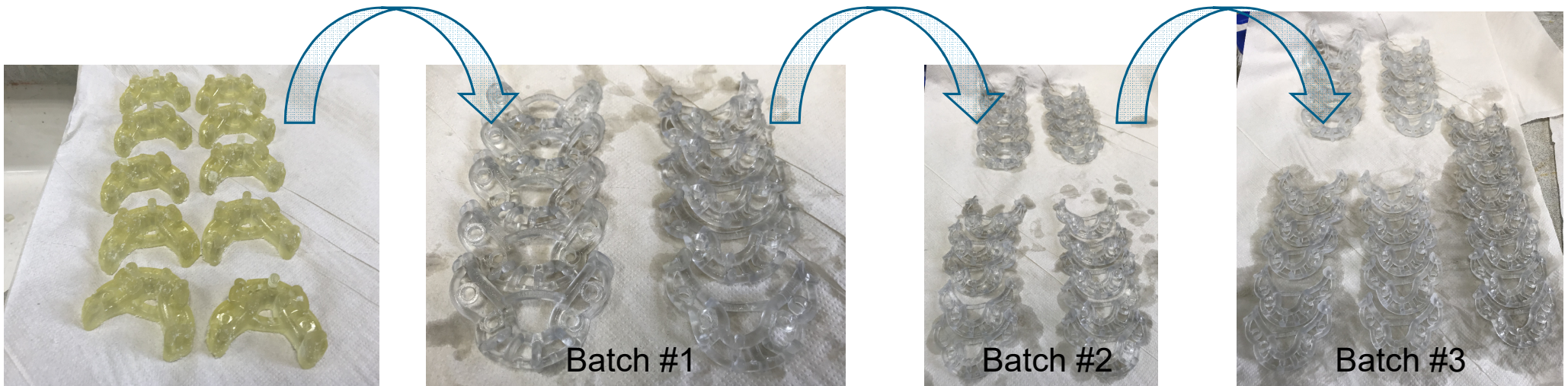

## Very Satisfactory Results


---



## C&B models in basket @60bar

Results:

- Cycle 12mins @ 60bar, both sides
  - 9 models tested
  - **0 broken**
  - **100% cleaned**

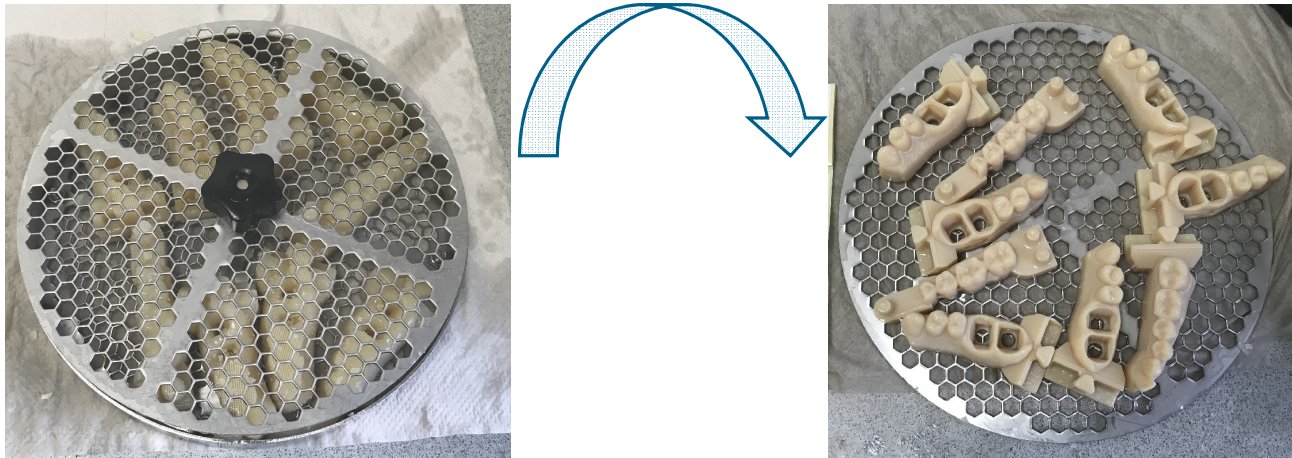

---

## C&B Models – Dies are well situated inside the clean model

---



## C&B models in basket @60bar

### Results:


- Choose the right model for your teeth
- Generally the ball basket is suitable for molded teeth, even if they are small.



# Implants @10bar

Results:

- Cycle 4mins @ 10bar
  - 10 models tested
  - 7 uncleaned
- Cycles 8mins @ 10bar
  - 14 models tested
  - 1 uncleaned

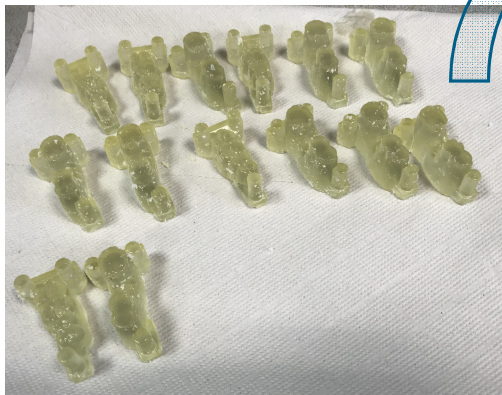







# Gingival Mask


Results:

- Cycles 12mins @ 20bar
  - 12 models tested
  - 1 uncleaned
  - 1 lost

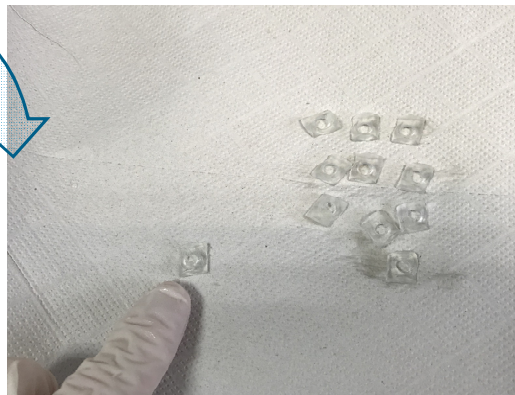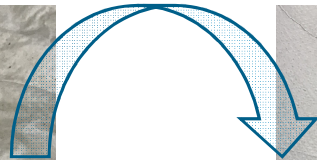

## Small Pieces – Choice of Proper Container

- Small pieces are smaller than the holes inside the tank
  - That enforces the usage of a designated container

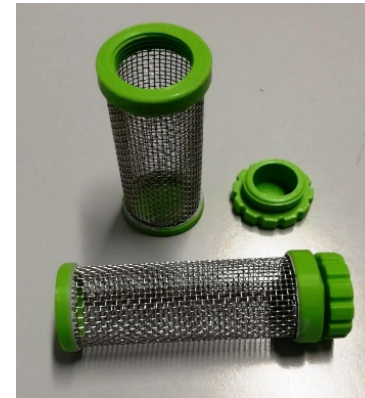

---

## Small teeth - Choice of appropriate container

---

Removal of the SUP705 from the molded parts.

In the case of small pieces based on their size and size, we recommend:

- Choice of the most suitable container for the parts to be washed.
- Search for the cycle time necessary to achieve total cleaning.
- Use pressure from 20 to 60 bar depending on the thickness of the pieces.
  
- Consider contacting your dealer for a solution by sending as much information as possible with photos and data from your work.


---

## Ortho Appliances – Small holes reachability

---

### Results:

- SUP705 is not removed completely from small holes within the model

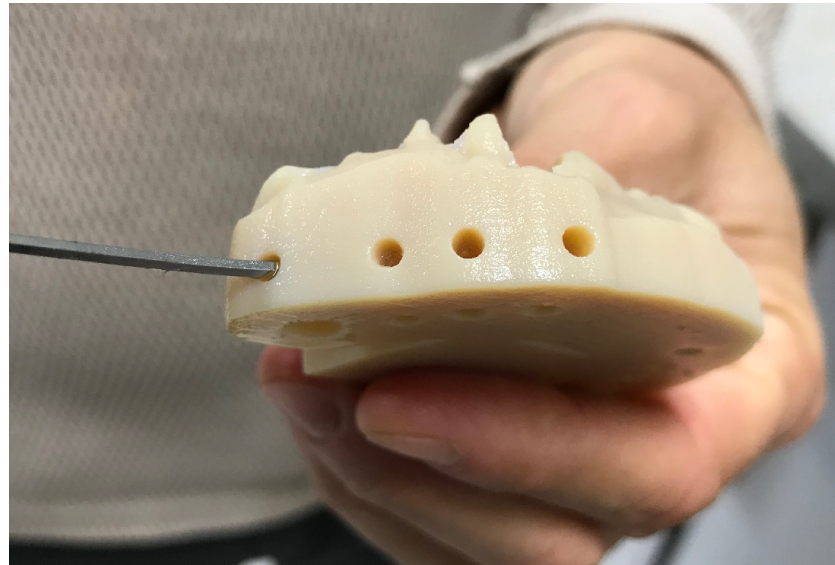

## Aligners w/706B @1.6mm Matte thickening

Results:

- Cycles 8 mins @ 20bar
  - 14 models tested
  - 0 broken (all cleaned)



THANK YOU

---

**stratasys**<sup>®</sup>