

Product	Max Thickness	Heraflash / UniXS / Hilite Power / Hilite Power 3D	Unilux AC	Spectramat	Triad 2000	Solidilite EX	Targis
---------	---------------	--	-----------	------------	------------	---------------	--------

Dentacolor XS

		SECONDS	CYCLES	MINUTES	MINUTES	MINUTES	MINUTES
Signum Bond II	Thin	90	15	5	6	7	6
Signum Bond I	0	0	0	0	0	0	0
Retention Flow	0.4	90	15	5	6	7	
Opaque	0.08	90	15	5	6	5	
Margin	2	90	5	2	2	7	
Composite Flow Dentine	2	90	15	5	6	7	
Dentine	2	90	15	5	6	7	
Enamel	2	90	5	5	2	7	
Effect	1	90	15	5	2	7	
Incisal	1	90	15	5	6	7	
Transpa	1	90	15	5	6	7	
Gingiva	2	90	15	5	6	7	
Creactive	2	90	15	5	6	7	25
Connector		90	5	2	2	5	6
Final Cure		180	25	10	20	9	