EN | PRO-FORM LAMINATE

Designed for a variety of intra-oral applications found in every dental office

1. After pouring up the model, remove the palatal area or thin out palatal region and drill a large hole.
2. Place laminate material into vacuum former so the Pro-Form name is up.
3. Raise frame to heat material
4. Allow the Pro-Form laminate to heat until it droops 1/2" below the vacuum formers frame.
5. Lower the Pro-Form laminate onto the model so that the Pro-Form palatal brace is approximately 3mm from the incisal edge on the lingual of the anteriors.
6. Vacuum form and allow material to cool at room temperature or place in cold water to cool.
7. Trim material off of the model with a crown and bridge scissors or Bard Parker blade.
8. For mouthguards, attach break-away strap at this point. (See strap attachment instructions).
9. Bevel edges with a large pear shape dual cut carbide bur or use a micro torch. Put guard on model when using a micro torch.
10. Use a Robinson Bristle Brush to smooth edges and feather out the gingival areas.
11. Final polishing can be done with a medium napped muslin buffing wheel.

Technical Note: For cleaning laminates, Keystone Industries recommends a periodic washing with a mild liquid soap and soft brush, followed by a rinsing in a pleasant flavored mouthwash to remove the soap taste. The laminate should then be stored in a well-ventilated plastic container when not in use.

For children and young adults: The fabricated laminates should be checked once a year by their dentists to insure proper fit.

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The Pro-Form laminate was originally developed by a dentist to provide superior intra-oral protection for professional hockey and football players. The unique two-layered construction provided retention, protection and comfort far beyond other materials. The Pro-Form laminate soon became the standard mouthguard materials used by professional sports players. Since this early beginning, dentists have reviewed, upgraded and modified the laminates and their usages in a variety of intra-oral applications.

Pro-Form is the only pre-made laminated Mouthguard material in the dental industry. Patented.

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