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Product Identification: **Splint Material** (EN)

Indicated Use: Hard/Thin Splint, Bleaching Tray, Orthodontic Retainers

Material: PETG
.020"(.5mm),.030"(.76mm), .040"(1mm), .060"(1.5mm), .080"(2mm), .100"(2.5mm) thickness

Available Variations: 5x5" Square (127mmx127mm), 120 mm & 125mm round

INSTRUCTIONS
REMOVE PROTECTIVE FILM PRIOR TO USE

Thinner materials do not require as much heat as thicker ones. The 020,030 & 040 thicknesses should be heated until approximately ¼" of droop occurs. Thicker materials should heat until approximately ½" of droop occurs.

After heating, vacuum form over the model.

Thinner materials can be trimmed using a scissor. Thicker materials can be trimmed using a wheel saw.

Storage: This material is hygroscopic (it absorbs moisture from the air). Moisture in the material will result in the formation of bubbles during the heating process. Always store the material in the packaging with dry gel pack provided.

Store the material in a cool, dry location.

Note: Material is not made with natural rubber latex, does not contain DEHP or other Phthalates, or Bisphenol A(BPA) and is gluten free.

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Keystone Industries
52 West King St.
Myerstown, PA 17067 USA
800.333.3131
keystoneindustries.com

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